



**NEOPHARMA
TECHNOLOGIES™**

DO YOU NEED FAST AND ACCURATE DRUG TEST RESULTS?

Welcome to the worlds leading Drugs of Abuse test kits empowered with

NEOVAULT™




CALL NOW TOLL FREE 1844 NEOTEST

enquiries@neopharmatechnologies.com

www.neopharmatechnologies.com



INSTRUCTIONS OF USE

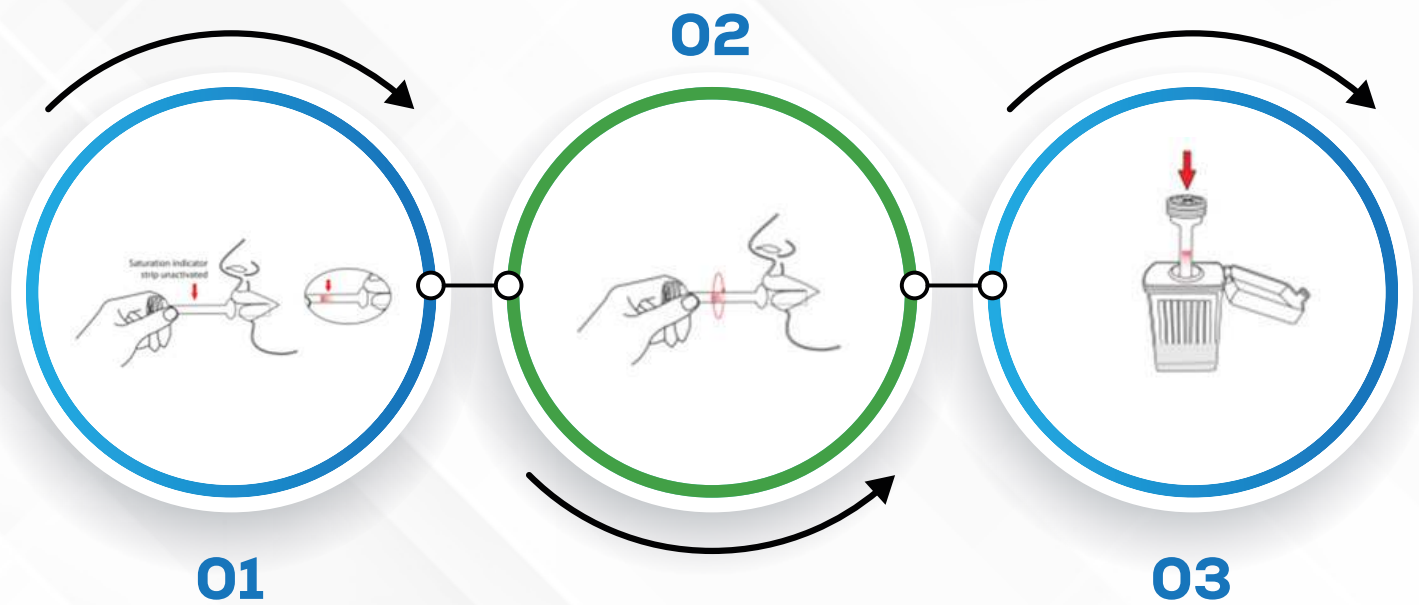

INTERPRETATION OF RESULTS



Product Description	Format	Cut-off Value	Qualification
7-Aminonclozepam (ACL) Test	Device	100 ng/mL	/
Amphetamine (AMP) Test	Device	50/40 ng/mL	CE
Barbiturates (BAR) Test	Device	300/50/30 ng/mL	CE
Benzodiazepines (BZO) Test	Device	50/20/10 ng/mL	CE
Buprenorphine (BUP) Test	Device	10/5 ng/mL	CE
Carisoprodol (SOMA) Test	Device	300 ng/mL	/
Cocaine (COC) Test	Device	50/20 ng/mL	CE
Codeine (COD) Test	Device	10 ng/mL	CE
Cotinine (COT) Test	Device	50/30/10 ng/mL	CE
Diphenhydramine (DIP) Test	Device	150/100 ng/mL	/
Ecstasy (MDMA) Test	Device	60/50 ng/mL	CE
Ethyl Glucuronide (EtG) Test	Device	150/100 ng/mL	/
Fentanyl (FEN) Test	Device	10 ng/mL	CE
Hydrocodone (HCD) Test	Device	10 ng/mL	/
Hydromorphone (HMO) Test	Device	300/150 ng/mL	/
Ketamine (KET) Test	Device	100/50 ng/mL	CE
Lysergic acid diethylamide (LSD) Test	Device	25/10 ng/mL	CE
Marijuana (THC) Test	Device	50/40/30/25/15/12/10/5/ 4/3 ng/mL	CE
Methadone Metabolite (EDDP) Test	Device	20 ng/mL	CE
Mephedrone (MEP) Test	Device	50 ng/mL	/
Methadone (MTD) Test	Device	75/50/30 ng/mL	CE
Methamphetamine (MET) Test	Device	50 ng/mL	CE

Product Description	Format	Cut-off Value	Qualification
Methaqualone (MQL) Test	Device	150/100 ng/mL	CE
Methcathinone (MTC) Test	Device	50 ng/mL	/
3,4-Methylenedioxypropylamphetamine (MDPV) Test	Device	200 ng/mL	CE
Methylphenidate (MPD) Test	Device	50 ng/mL	/
6-Monoacetylmorphine (6-MAM) Test	Device	25/15/10/5/4 ng/mL	CE
Morphine (MOP) Test	Device	15 ng/mL	CE
Opiate (OPI) Test	Device	50/40 ng/mL	CE
Oxycodone (OXY) Test	Device	50/40/20 ng/mL	CE
Phencyclidine (PCP) Test	Device	10 ng/mL	CE
Phenytoin (PHEN) Test	Device	150/100 ng/mL	/
Pinaca Ab (K3) Test	Device	10 ng/mL	/
Pregabalin (PGB) Test	Device	100 ng/mL	/
Propoxyphene (PPX) Test	Device	50/20 ng/mL	CE
Synthetic Marijuana (K2) Test	Device	25/10/5 ng/mL	CE
Tramadol (TRA) Test	Device	100/50 ng/mL	CE
Tricyclic Antidepressants (TCA) Test	Device	100 ng/mL	CE
XLR-11 Test	Device	100 ng/mL	/
Zolpidem (ZOL) Test	Device	25 ng/mL	/
Zopiclone (ZOP) Test	Device	25 ng/mL	/
Alcohol (ALC) Test	Device	0.05/0.02%	CE



INSTRUCTIONS OF USE



2 MINUTES

	Normal	Abnormal
OXI	Negative	
SG	Low, Normal, High	Low, High
pH	4.5, 5.5, 6.5	7.5, 8.5
NIT	Negative	Target
GLUT	Negative	Target
CREA	Target	Target



4 MINUTES

Urine Alcohol Color			
0	40	80	200
0.00%	0.04%	0.08%	0.20%




5 MINUTES






INSTRUCTIONS OF USE



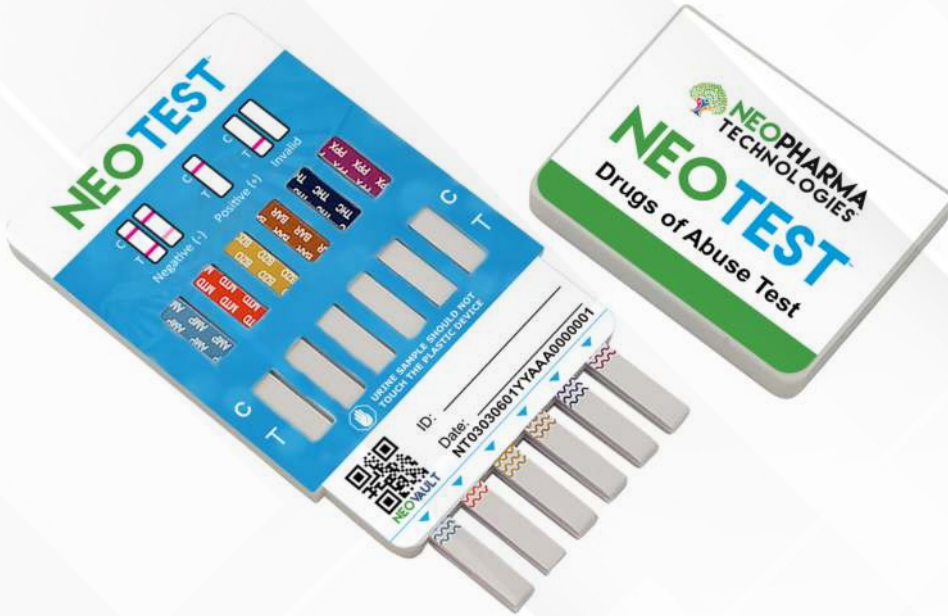

INSTRUCTIONS OF USE

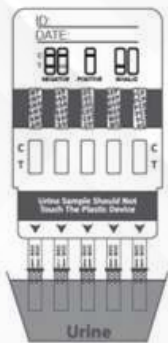

2 MINUTES

	Normal	Abnormal
OXI	Negative	Positive
BG	Normal	Abnormal
pH	Normal	Abnormal
NIT	Negative	Positive
GLUT	Negative	Positive
CREA	Normal	Abnormal



4 MINUTES

Urine Alcohol Color			
0	45	90	225
0.0%	0.04%	0.08%	0.20%



5 MINUTES



Product Description	Format	Cut-off Value	Qualification
Acetaminophen (ACE) Test	Strip/Cassette/Dip Card/Cup	5000 ng/mL	CE
Amphetamine (AMP) Test	Strip/Cassette/Dip Card/Cup	2000/1000/500/300/250 ng/mL	CE 510(k)
Barbiturates (BAR) Test	Strip/Cassette/Dip Card/Cup	2000/600/300/200 ng/mL	CE 510(k)
Benzodiazepines (BZO) Test	Strip/Cassette/Dip Card/Cup	600/400/300/200/100 ng/mL	CE 510(k)
Buprenorphine (BUP) Test	Strip/Cassette/Dip Card/Cup	10/5 ng/mL	CE 510(k)
Carisoprodol (SOMA) Test	Strip/Cassette/Dip Card/Cup	1000 ng/mL	CE
Clonazepam (CLO) Test	Strip/Cassette/Dip Card/Cup	500/100 ng/mL	CE
Cocaine (COC) Test	Strip/Cassette/Dip Card/Cup	600/300/150/100 ng/mL	CE 510(k)
Codeine (COD) Test	Strip/Cassette/Dip Card/Cup	2000 ng/mL	CE
Cotinine (COT) Test	Strip/Cassette/Dip Card/Cup	400/300/200/100/50 ng/mL	CE
Ecstasy (MDMA) Test	Strip/Cassette/Dip Card/Cup	2000/1000/500/300/250/150 ng/mL	CE 510(k)
Ethyl Glucuronide (EtG) Test	Strip/Cassette/Dip Card/Cup	500/300ng/mL	CE
Fentanyl (FEN) Test	Strip/Cassette/Dip Card/Cup	300/200/100/50 ng/mL	CE
Norfentanyl (FEN) Test	Strip/Cassette/Dip Card/Cup	200/50/20/10/5 ng/mL	CE
Gabapentin (GAB) Test	Strip/Cassette/Dip Card/Cup	3750/2000/1000 ng/mL	CE
Hydrocodone (HCD) Test	Strip/Cassette/Dip Card/Cup	300/10 ng/mL	CE
Hydromorphone (HMO) Test	Strip/Cassette/Dip Card/Cup	300 ng/mL	CE
Ketamine (KET) Test	Strip/Cassette/Dip Card/Cup	3000/2000/1000/500/100 ng/mL	CE
Kratom (KRA) Test	Strip/Cassette/Dip Card/Cup	250/150/100 ng/mL	CE
Lysergic acid diethylamide (LSD) Test	Strip/Cassette/Dip Card/Cup	20 ng/mL	CE
Marijuana (THC) Test	Strip/Cassette/Dip Card/Cup	600/300/200/150/50/40/25/20/18/15 ng/mL	CE 510(k)
Methadone Metabolite (EDDP) Test	Strip/Cassette/Dip Card/Cup	300/100 ng/mL	CE 510(k)
Methadone (MTD) Test	Strip/Cassette/Dip Card/Cup	1000/600/300/200/50 ng/mL	CE 510(k)
Methamphetamine (MET) Test	Strip/Cassette/Dip Card/Cup	2000/1000/500/300/250 ng/mL	CE 510(k)
Methaqualone (MQL) Test	Strip/Cassette/Dip Card/Cup	300/1000 ng/mL	CE
Methcathinone (MTC) Test	Strip/Cassette/Dip Card/Cup	500/300 ng/mL	CE
3,4-Methylenedioxypropylvalerone (MDPV) Test	Strip/Cassette/Dip Card/Cup	1000/500/300 ng/mL	CE
Methylphenidate (MPD) Test	Strip/Cassette/Dip Card/Cup	300 ng/mL	CE
6-Monoacetylmorphine (6-MAM) Test	Strip/Cassette/Dip Card/Cup	20/10 ng/mL	CE
Morphine (MOP) Test	Strip/Cassette/Dip Card/Cup	2000/600/300/150/100 ng/mL	CE 510(k)
Opiate (OPI) Test	Strip/Cassette/Dip Card/Cup	2000/300/100 ng/mL	CE 510(k)
Oxycodone (OXY) Test	Strip/Cassette/Dip Card/Cup	300/100 ng/mL	CE 510(k)
Phencyclidine (PCP) Test	Strip/Cassette/Dip Card/Cup	50/25 ng/mL	CE 510(k)
Pinaca Ab (K3) Test	Strip/Cassette/Dip Card/Cup	10 ng/mL	CE
Pregabalin (PGB) Test	Strip/Cassette/Dip Card/Cup	2000/1000/500 ng/mL	CE
Propoxyphene (PPX) Test	Strip/Cassette/Dip Card/Cup	600/300 ng/mL	CE 510(k)
Synthetic Marijuana (K2) Test	Strip/Cassette/Dip Card/Cup	75/50/25/20/10 ng/mL	CE
Tramadol (TRA) Test	Strip/Cassette/Dip Card/Cup	200/100 ng/mL	CE
Tricyclic Antidepressants (TCA) Test	Strip/Cassette/Dip Card/Cup	1000/300 ng/mL	CE 510(k)
UR-144 Test	Strip/Cassette/Dip Card/Cup	50 ng/mL	CE
Zolpidem (ZOL) Test	Strip/Cassette/Dip Card/Cup	50 ng/mL	/
Zopiclone (ZOP) Test	Strip/Cassette/Dip Card/Cup	50 ng/mL	/
Alcohol (ALC) Test	Strip/Cassette/Dip Card/Cup	0.04%	C